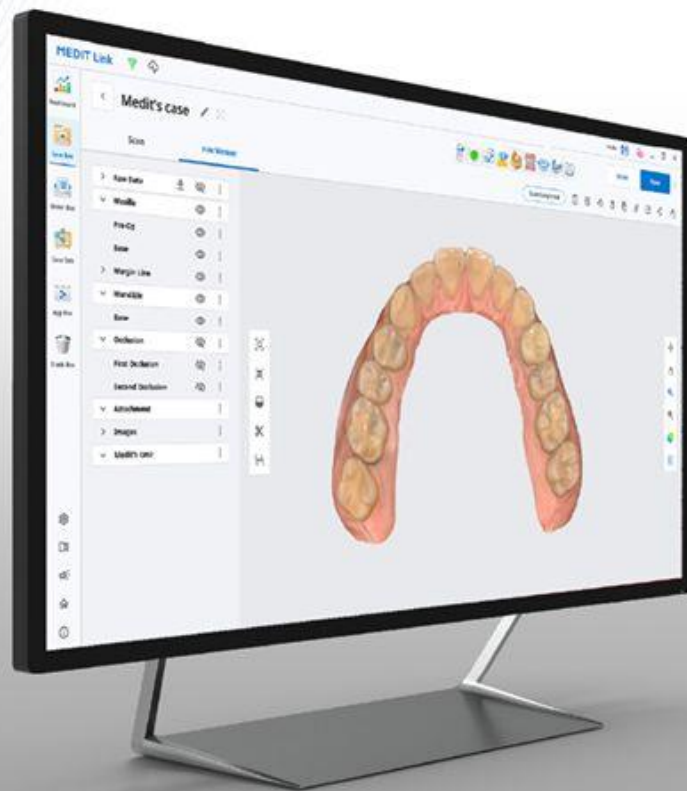
**MEDIT Link** + **Carestream**  
DENTAL

**MEDIT**

Medit

9,582 follower

2s • 🌐

We are excited to share with you that Carestream Dental's CS Imaging version 8 has been integrated with Medit Link!

Users can now open up Medit Link from CS Imaging, scan, and then view the scan data from CS Imaging directly.

Carestream Dental's CS Imaging version 8 centralizes and displays all 2D and 3D images, along with CAD/CAM data, in one convenient location to aid in more comprehensive diagnoses, optimized treatment workflows and enhanced case presentation.

Please reach out to a local Carestream Dental representatives for any inquiries.

If you wish to know more, please click the link below:

[https://hubs.la/Q01BWH\\_D0](https://hubs.la/Q01BWH_D0)

#Medit #Meditlink #Carestreamdental #CSImagingv8 #Integration #PACS  
#scandata #Imaging #digitaldentistry

MEDIT

- Dashboard
- Case Box
- Order Box
- Case Talk
- App Box
- Trash Box

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[Learn more >](#)

**Medit i600**

The Brain of the i700 with a Splash of Color

[Learn more >](#)

Check that the programme is correctly installed


The screenshot shows a user interface for an application marketplace. At the top, there are two filters: 'Tutte le app' (All apps) and 'Tutte le categorie' (All categories). Below this, the 'CS Imaging Integration' app is displayed. It features a logo, a category tag 'Socio', and sub-categories 'PACS/PMS' and 'Software di imaging'. The developer is 'Di Carestream Dental', the version is 'v 1.0.0.0', and the release date is '2023-02-10'. Below the app details, it shows 'Download totali 220' (Total downloads 220) and a grey button labeled 'Installato' (Installed). A red box highlights the 'Installato' button, and a green arrow points from the text on the left towards it. Below the first app, the 'fastscan.io Integration' app is partially visible, with a logo, category tag 'Strumento', developer 'Di Medit R&D', version 'v 1.0.1.0', and release date '2022-12-16'.

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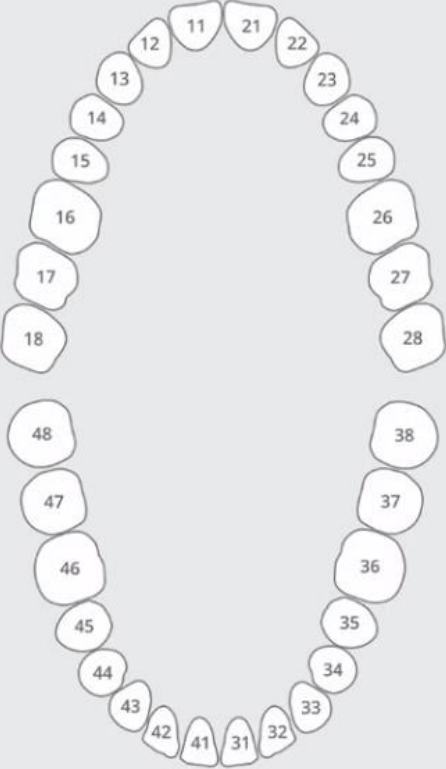Elenco degli ordini

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Annulla Salva

contrary to the other apps, you will not see the Carestream icon at the top of the bar.

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


SCAN

Data | Tipo | Filtri

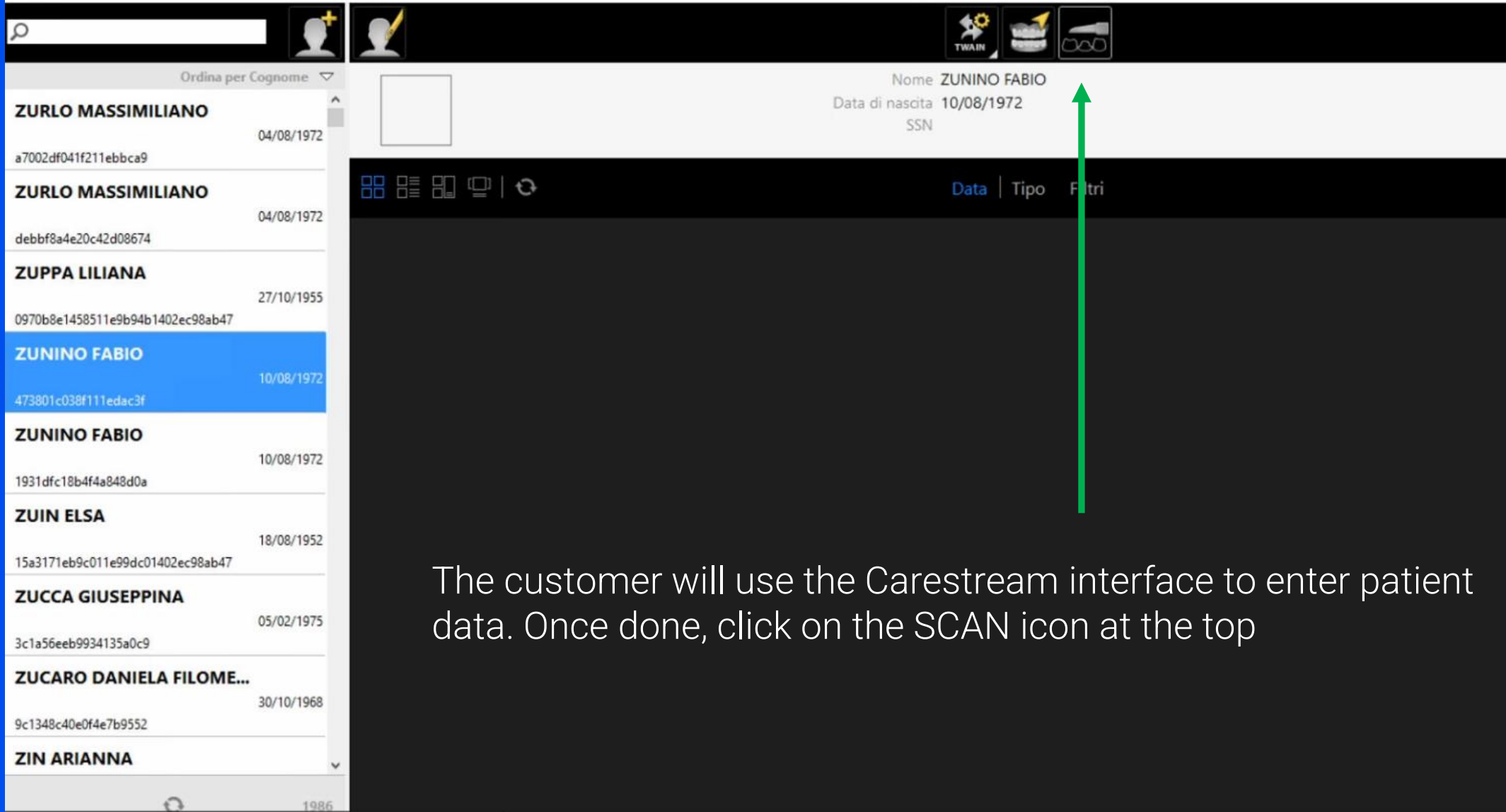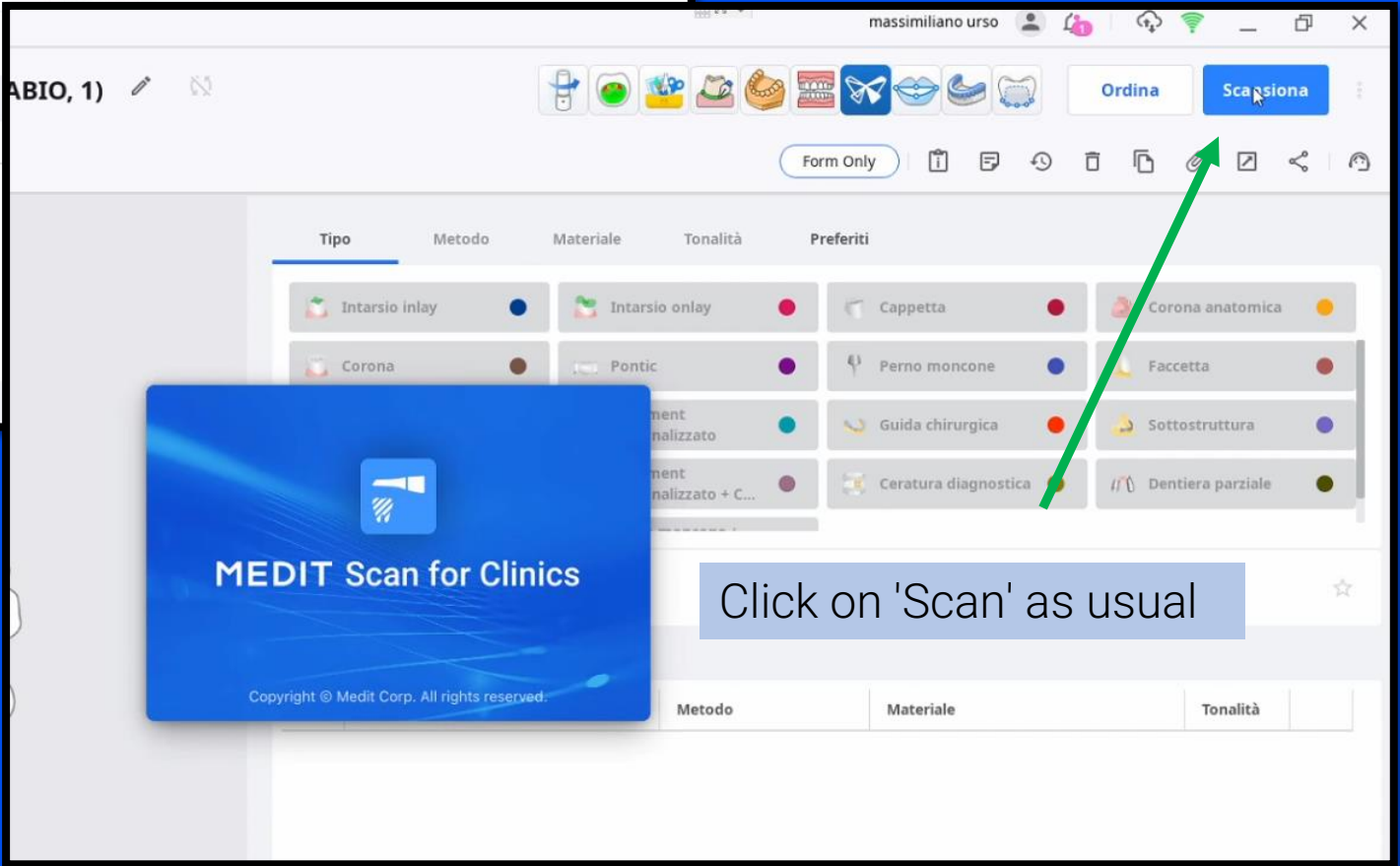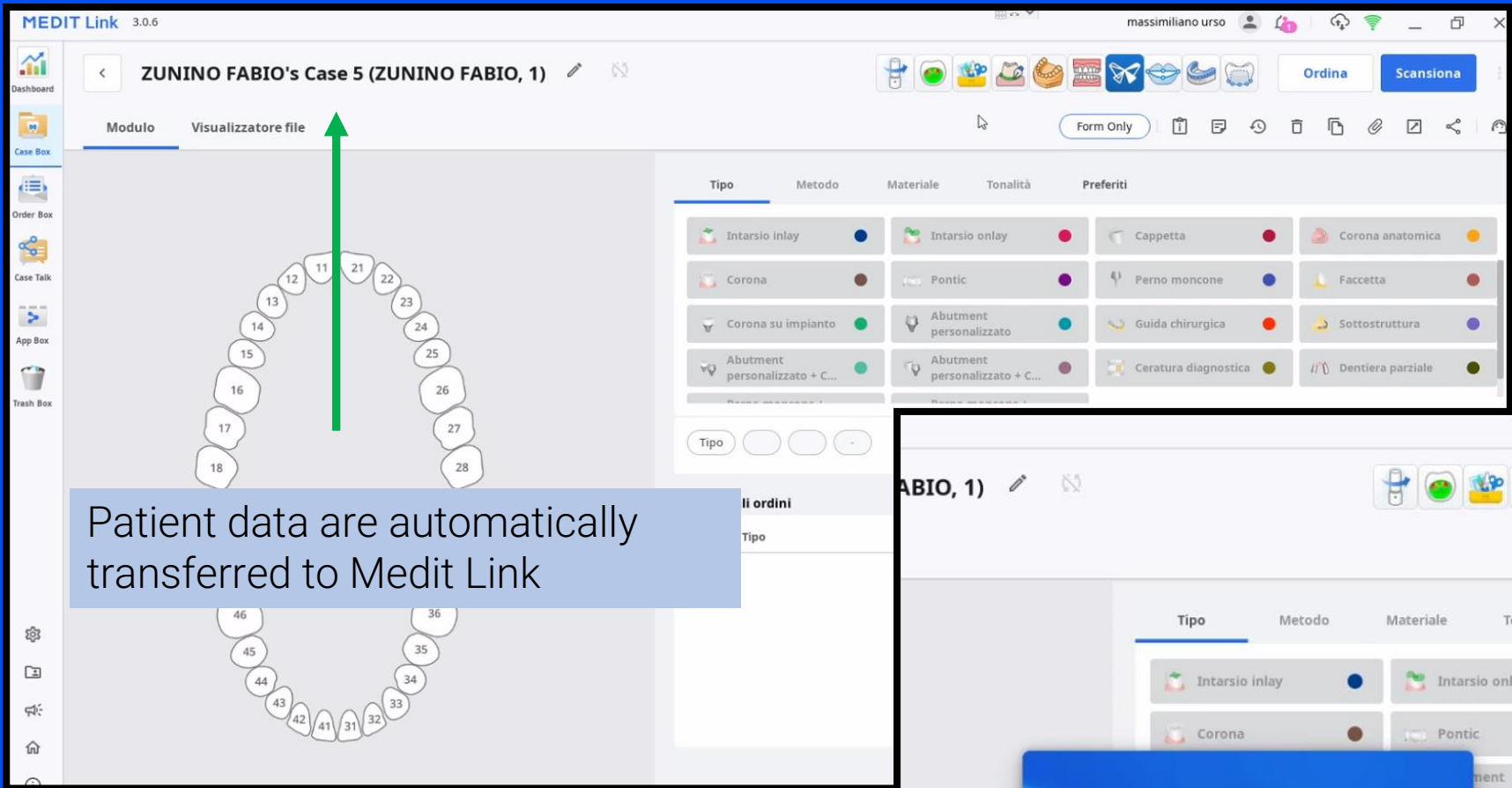The customer will use the Carestream interface to enter patient data. Once done, click on the SCAN icon at the top

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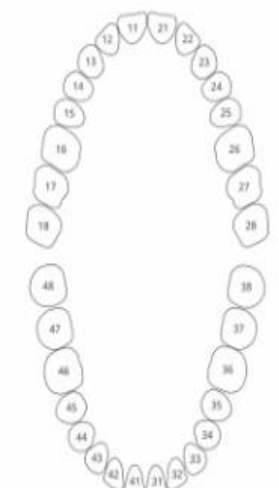
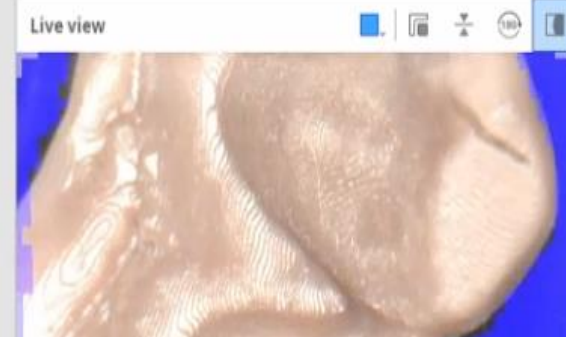
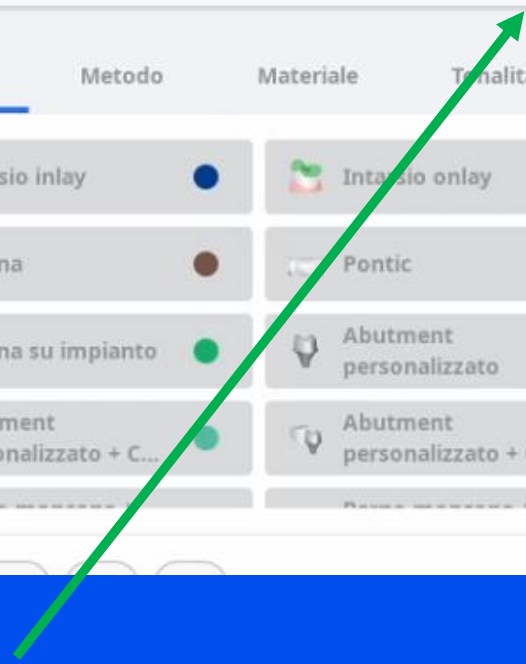
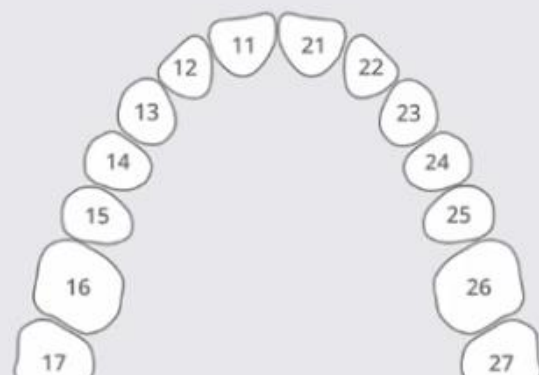
Wait until the end of the processing stage

Search bar with magnifying glass icon and a user profile icon with a plus sign.

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Data | Tipo | Filtri

Importa modello

Importing is in progress, please wait...

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Once the process is finished, the file is automatically uploaded to the CS Imaging platform

Search bar with a magnifying glass icon and a user profile icon with a plus sign.

User profile icons: a user profile with a pencil and a TWAIN scanner icon.

Ordina per Cognome ▾

**ZURLO MASSIMILIANO**  
04/08/1972  
a7002df041f211ebbca9

**ZURLO MASSIMILIANO**  
04/08/1972  
debbf8a4e20c42d08674

**ZUPPA LILIANA**  
27/10/1955  
0970b8e1458511e9b94b1402ec98ab47

**ZUNINO FABIO**  
10/08/1972  
473801c038f111edac3f

**ZUNINO FABIO**  
10/08/1972  
1931dfc18b4f4a848d0a

**ZUIN ELSA**  
18/08/1952  
15a3171eb9c011e99dc01402ec98ab47

**ZUCCA GIUSEPPINA**  
05/02/1975  
3c1a56eeh003d135a0r0

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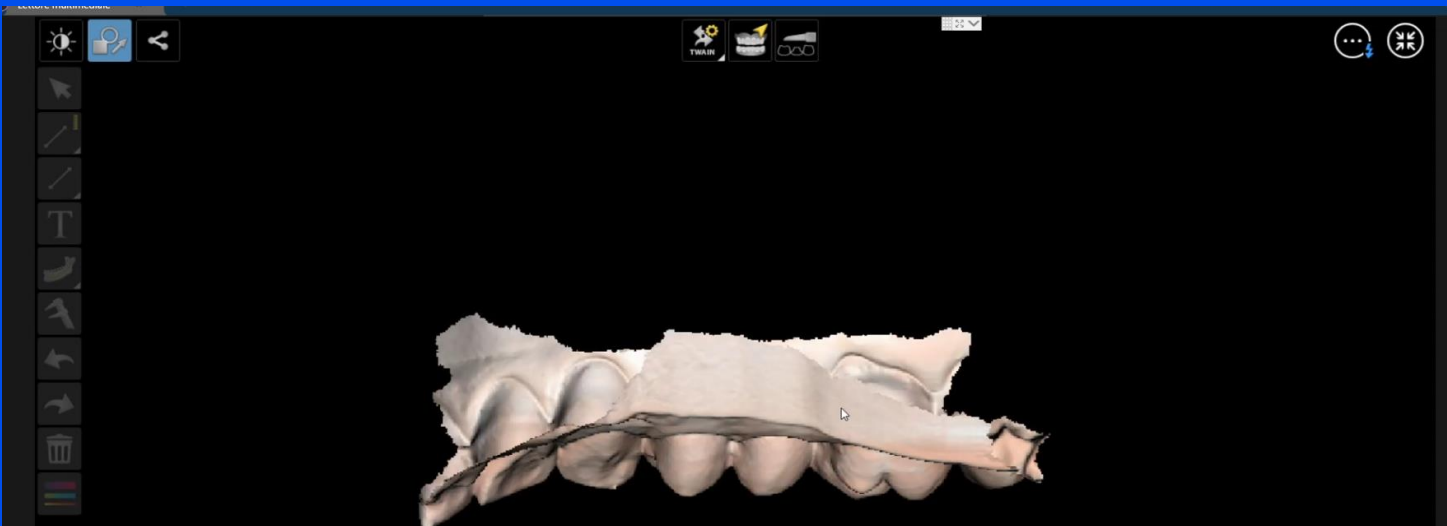
Data


28/02/2023 13:44

Once uploaded, the file will be available for use.

Carestream 3D viewer

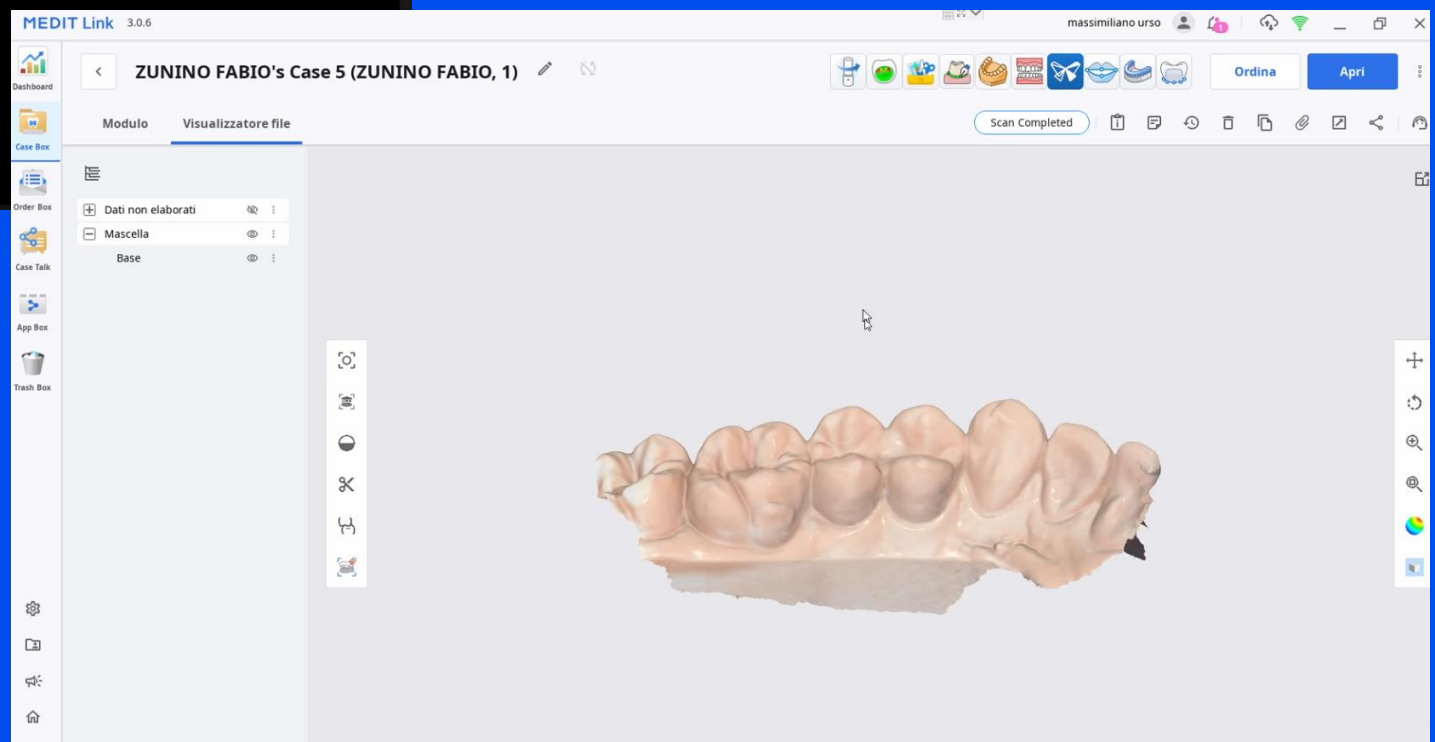Medit 3D viewer

MEDIT

Thank you!

Scanning Reimagined